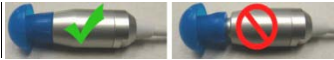
Quick Reference Guide – ERO•SCAN

Powering the Unit

1. Press **DOWN ARROW**↓ to turn unit **ON**.
2. The test will default to the last protocol selected.
3. Press and hold **UP ARROW**↑ to turn unit **OFF**.

Placing the Probe

1. Make sure ear canal is clear of debris.
2. Select a colored silicone eartip and place on probe tube, fully seating it on the probe.

3. Insert probe in the ear. Probe should remain in place without hand-holding.

Changing the Probe Tube

NOTE: Probe tubes do NOT have to be changed for every test, but may need to be replaced if clogged.

1. To remove probe tube, flip eartip up and use to grip tube.

2. Twist slightly and pull tube straight out of probe head.
3. Insert a new probe tube into the probe head until it is fully seated.

Conducting a Test

1. Press **CHANGE**↓ to select a different protocol.

2. Press **CHANGE**↔ to scroll to desired protocol.


3. Press **UP ARROW**↑ to return to test screen.
4. Press **<L** or **R>** to select test ear and begin test.

5. Probe Check will appear on screen. The scale should be in the green area. If the scale is in the amber area, reposition the probe in the ear.

6. Test will automatically start once probe check is acceptable.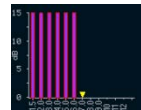

7. When test is finished, Press **<L** or **R>** to test the other ear, or press **REVIEW**↓ to review results graph.

NOTE: "Pass" or "Refer" will only be displayed if set in protocol.

Printing Test Results via Ero•Scan Database

1. Open ERO•SCAN Database on PC.
2. Connect ERO•SCAN unit to computer via included USB/micro-USB cord.
3. Make sure ERO•SCAN unit is turned on.
4. "Waiting on PC" will display on ERO•SCAN screen.
5. If this is a new or different ERO•SCAN device,

click **"Detect Com Port"**.

6. Click **"Results"**.
7. Results in the ERO•SCAN unit will download to database.
8. Select the results you would like to print

Result						Equipment	
Test #	Ear	Test Date	Outcome	Protocol	Tester	Notes	Instrument SN. Probe
2	Left	3/4/2012 1:27:15 PM		DP 1.6-8.0			ME3000001 MEdemo35
1	Right	3/4/2012 1:26:06 PM		DP 1.6-8.0			ME3000001 MEdemo35
4	Left	3/4/2012 1:44:21 PM	PASS	DP 2.0-5.0			ME3000001 MEdemo35
3	Right	3/4/2012 1:43:51 PM	PASS	DP 2.0-5.0			ME3000001 MEdemo35
14	Right	3/5/2012 10:08:17 AM	PASS	DP 2.0-5.0			ME3000001 MEdemo35

9. Click **"Print"**.

NOTE: For additional functions and instructions on using the ERO•SCAN database, please see the included **Database Operation Manual**.

NOTE: For additional information on how to print to the optional Bluetooth thermal printer, please see the included **ERO•SCAN Operation Manual**.

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