








Quick Reference Guide – easyTym™




Powering the Unit


1. Press the **RED** or **BLUE ARROW**   to turn the unit **ON**.
2. The test will default to the protocol last selected.
3. Press and hold the **RED and BLUE ARROWS**   together to turn the unit **OFF**.

Conducting a Test

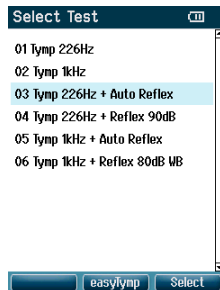1. Press **SELECT TEST**  on left side of screen.
2. Use **UP** and **DOWN ARROWS**  to highlight desired test protocol.



3. Press **SELECT** on the right side of screen.
4. Select test ear using the **RED (RIGHT)** or **BLUE (LEFT) ARROW** keys.  
5. Test will start when probe is placed in the ear. Status of the test is shown at the top of screen.
6. When test is finished, use **UP** or **DOWN ARROWS**  to scroll through screen and review results.

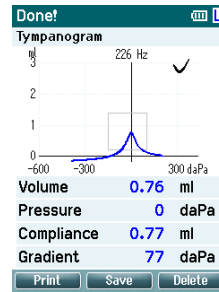

NOTE: If “show pass/fail” is turned on in the set-up, a ✓ (check mark) indicates a pass, and an X indicates a refer result.

Printing Test Results to Thermal Printer

1. To print results to the thermal printer, turn the printer on, wait for printer icon to display in top right corner of screen, press **PRINT** button.

Saving Test Results to NOAH Module

1. Make sure cradle is connected to computer via USB cord.
2. Open MAICO Impedance module via NOAH database.






NOTE: Upon initial set-up of the software, click on **EXTRAS** and select **SETTINGS**. Select **USB (easyTym Communicator)**. Then click **OK**.




3. Place unit in cradle to download test results.

NOTE: If results do not download automatically, click **REQUEST MEASUREMENT** at top of software module screen.

Set-Up Menu

1. Press **SELECT TEST** .
2. Press **EASYTYMP**.
3. Scroll with **UP** and **DOWN ARROWS**  to highlight a Set-up Menu selection.
4. Press **SELECT**.
5. Use **UP** and **DOWN ARROWS**  to scroll.
6. Press **RIGHT** or **LEFT ARROWS** to change selection.  

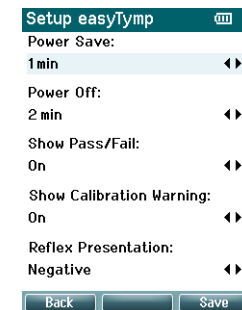

7. Press **SAVE** to save all changes and exit back to Set-up Menu screen.
8. Press **BACK** to exit back to protocol list.